



User Guide for the **XDcuff®**

PRODUCT REF 104

REUSABLE SECUREMENT STRAPS



The **XDcuff® Cuff Securement Straps (CSS)**, were developed to function as part of a system that enables you to quickly limit a patient's movement, on the stretcher, should the need exist.

Made from black nylon with DuraGrip® hook & loop fasteners, the **CSS** attach to the stretcher's frame. Once in position, they secure the **XDcuff®** disposable or reusable restraints, (**Product REF 102, 103, 101**) to the stretcher and store them for rapid access.


INSTALLATION OVERVIEW

Attaching the Cuff Securement Strap to Stretcher-

At the four recommended locations (*fig-3*), wrap the end of the **gray stretcher strap** around the **stretcher frame** (*as indicated*

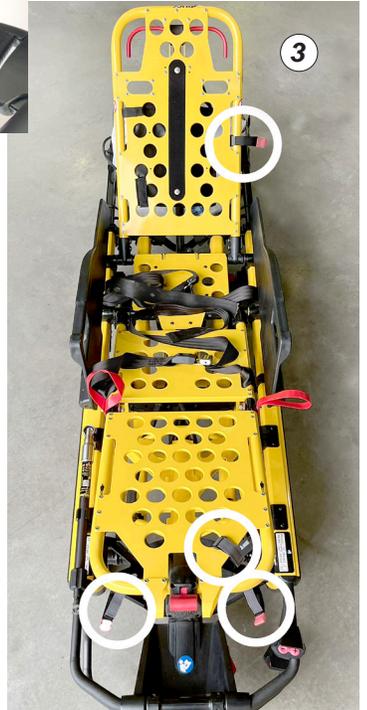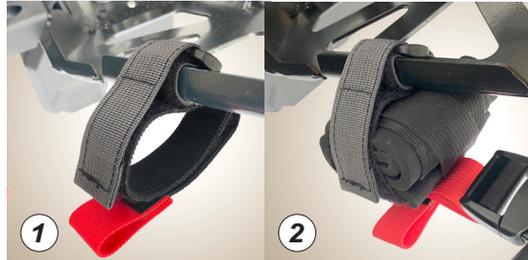


CUFF SECUREMENT STRAP (CSS)

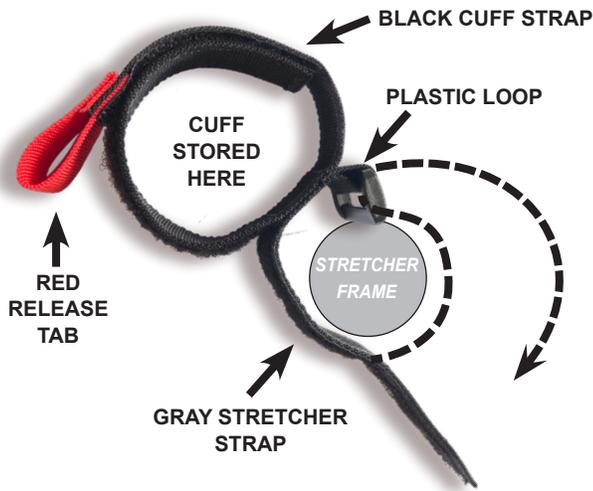

in diagram at left), feed it through the plastic loop and back on to itself. Smooth the hoop & loop fastener to secure (*fig-1*). When attaching the restraint to the **CSS**, pull the **red release tab** on the black cuff strap and open the hook & loop fastener. In the opening, place a folded (or packaged) restraint and re-attach the **cuff strap**. Smooth the hook & loop to hold the restraint in place. Your **XDcuff®** restraint is now ready for use (*fig-2*).

LEFT ANKLE POSITION - The left ankle position is located on the foot-end hitch, at the patient's left side toward the foot. Ensure that each **red release tab** is pointing towards the lift bars/handles for easy access in emergencies.

1) Wrap the gray stretcher anchor around the foot-end hitch (*fig-4*). **2)** Feed the **gray stretcher strap** through the **plastic loop** (*fig-5*). **3)** Pull the **gray stretcher strap** tightly and fasten it back to itself (*fig-6*). Completed left ankle position with **black cuff strap** secured (*fig-7*).

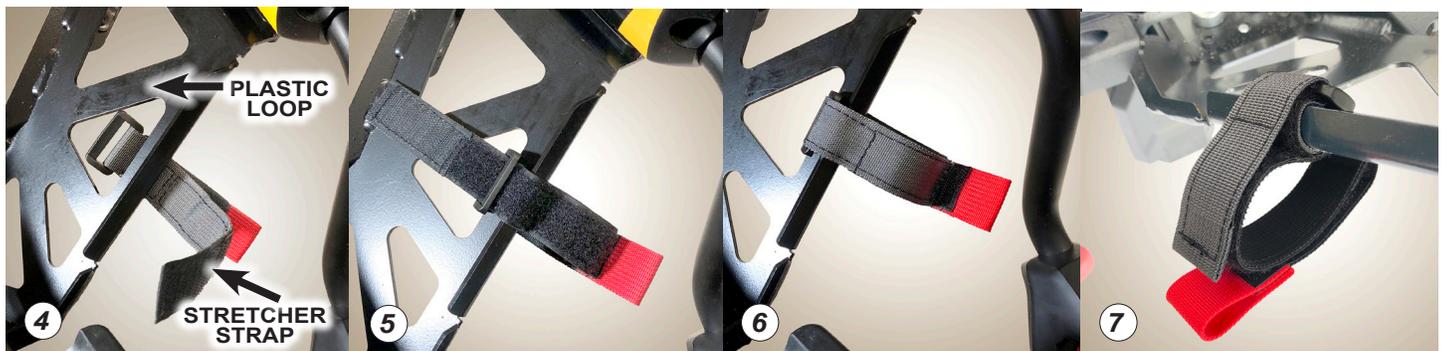

See User Guides packaged with the Restraints for detailed instructions on installing the **XDcuff®** system.

DISTRIBUTED BY DAVES MEDICAL LLC

david@xdcuff.com | www.xdcuff.com | 386.275.8361


XDcuff® - MADE IN THE USA
Patent Pending

CUFF SECUREMENT STRAP - WRIST & ANKLE POSITIONS


RIGHT ANKLE POSITION - The right ankle position is located on the opposite side of the foot-end-hitch.

1) Wrap the **gray stretcher strap** around the foot-end hitch (*fig-8*) **2)** Feed the **gray stretcher strap** through

the plastic loop (*fig-9*) **3)** Pull the gray stretcher strap tightly, and fasten it back to itself (*fig-10*). Completed left and right ankle position with **black cuff straps** secured (*fig-11*).

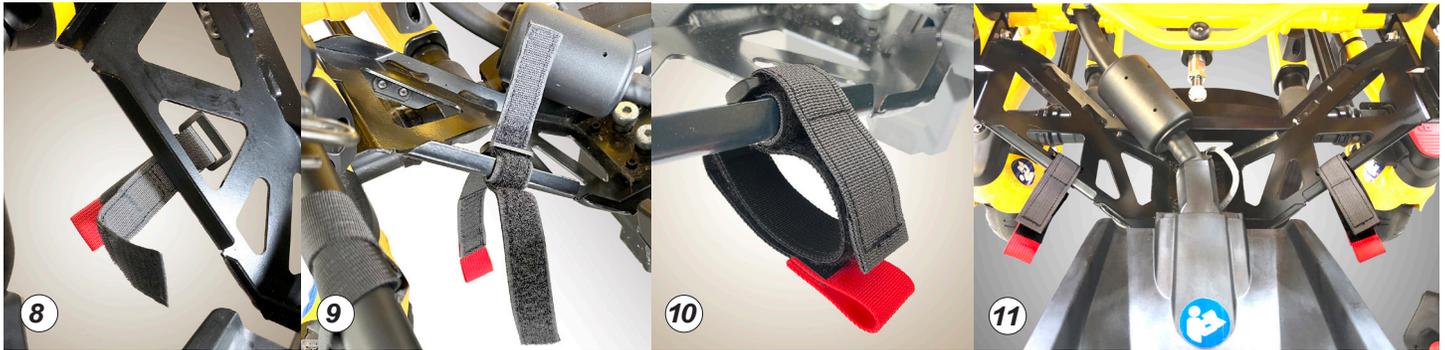

LEFT WRIST POSITION - This position is located on the foot lift portion of the stretcher, near the patients left foot.

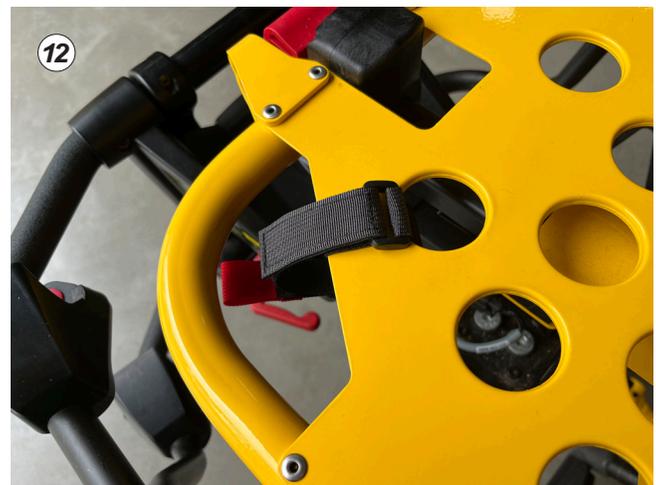

RIGHT WRIST POSITION - This position is located on the head lift portion of the stretcher on the patients left side.



Visit our YouTube channel (XDCuff®) for helpful instructional videos for installing the Cuff Securement Straps, Wrist and Ankle Restraints.